Instructions for use

FOOT-UP®

07810-X


07811-X

27810-X

27811-X

Life Without Limitations®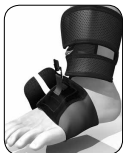

| | | |
|----|---------------------------------|----|
| | | 3 |
| ⓔⓃ | Instructions for use | 4 |
| ⓓⓔ | Gebrauchsanweisung | 5 |
| ⓕⓚ | Notice d'utilisation | 6 |
| ⓔⓢ | Instrucciones para el uso | 7 |
| ⓔⓣ | Istruzioni per l'uso..... | 8 |
| Ⓝⓞ | Bruksanvisning | 9 |
| ⓓⓐ | Brugsanvisning | 10 |
| ⓢⓕ | Bruksanvisning | 11 |
| ⓕⓙ | Käyttöohjeet | 12 |
| Ⓝⓕ | Gebruiksaanwijzing | 13 |
| ⓓⓣ | Instruções de Utilização..... | 14 |
| ⓙⓐ | 取扱説明書..... | 15 |
| ⓗⓗ | 中文说明书..... | 16 |
| Ⓚⓞ | 사용 설명서 | 17 |

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Foot-raising paresis

WARNING

In the case of sensitive skin or when at risk of skin ulcerations and infections, such as in the case of diabetes or peripheral vascular disease, caution is needed and you should consult your physician before using the product.

FITTING THE BASIC VERSION

The Foot-Up basic version can only be positioned and used in lace-up shoes.

- Measure the circumference of the ankle around the malleoli (**Figure A**).
- Detach the inlay from the ankle strap by pressing the button in the middle of the quick-acting closure.
- Apply the ankle strap with the narrow Velcro strip at the bottom and the quickacting closure at the front. In doing this place the ankle strap above the widest part of the ankle. Fasten the ankle strap tightly, but do not constrict the ankle (**Figure B**).
- Place the inlay between the tongue of the shoelaces. Then pull the elastic strap between the two lace lines (**Figure C**). By pulling the inlay further in the direction of the front of the foot more support is provided when raising the foot. Make sure the elastic strap is also pulled between the lace lines further down the foot.
- Connect the ankle strap and the inlay by pushing the two parts of the quick-acting closure together (**Figure D**).

FITTING THE SHOELESS VERSION

- Measure the circumference of the foot immediately behind the ball of the foot (**Figure E**).
- Apply the ankle strap with the narrow Velcro strip at the bottom and the quickacting closure at the front. In doing this place the ankle strap above the widest part of the ankle. Fasten the ankle strap tightly, but do not constrict the ankle (**Figure F**).
- Place the shoeless bandage immediately behind the ball of the foot, so that the button rests on top of the foot (**Figure G**).
- Connect the ankle strap and the inlay by pushing the two parts of the quick-acting closure together (**Figure H**).

WASHING & CARE

The liner should be washed by hand using a mild detergent and 30°C (86°F) cold water. Hang to dry. Do not put the Foot-Up in a washing machine or dryer.

LIST OF MATERIALS

Polypropylene, Polyester, Spandex, Nylon, Cotton, PVC

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